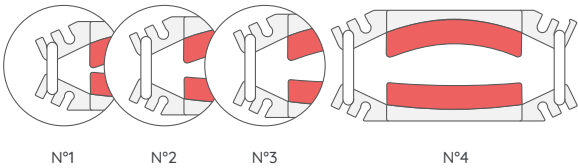
Stop-uri®

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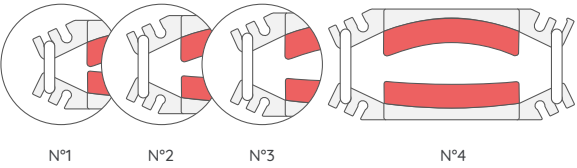
Stop-uri®

The Stop-uri kit consists of 2 clamps, each with 4 pairs of adjustment clips. 2 spare pairs of each size are also provided in case of loss.

Clip Sizes

Using the 4 clip sizes allows you to vary the spacing between the top and bottom.


Fitting

Place the clamp just behind the glans, without retracting the foreskin, with the penis well centered. The flat foam goes underneath; the rounded part goes on top. Make sure the skin and the clamp are clean and dry. Choose the clips that compress the urethra without pain or discomfort, and without creasing the skin. You may wear a small absorbent pad: slight leakage may persist (especially during physical effort).

Use

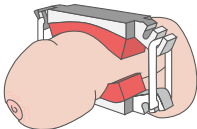The Penile clamp can be worn for several consecutive hours. Remember to urinate regularly to avoid excessive bladder filling. The width of the device helps maintain blood circulation, which continues naturally along the sides. The memory foam regains its softness: alternate the two clamps every other day to preserve comfort.


Practical Tips

Support is better when wearing briefs. Be attentive when moving from sitting to standing, as the clamp may slide slightly. Overweight men may feel discomfort if the clamp presses into the skin, particularly at the thighs; in this case, place a thin piece of fabric between the clamp and the skin for extra comfort. At night, leakage is often reduced; preferably wear an absorbent protection. If you keep the clamp on at night, choose larger clips only if you no longer have nocturnal erections.

Assembly and Care

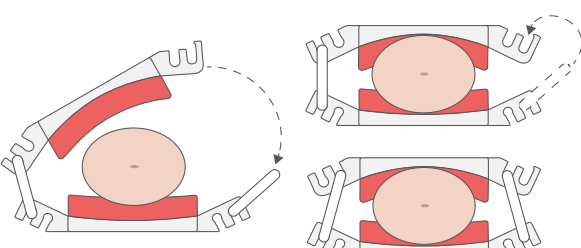One clamp is supplied pre-assembled as an example. Assemble the second with the clips appropriate to your size, arranged vertically or at an angle according to comfort. Clean with water and mild soap, no abrasive products, then air-dry before reuse.

Important

In case of skin irritation, a gauze pad can be placed between the Stop-uri clamp and the skin. Do not use in cases of acute urinary tract infection or lesions on the penis. Do not use the product if you have a known intolerance to the materials listed.

Warranty

The Stop-uri clamp is covered by a 2-year warranty against manufacturing defects (under normal, recommended use), foam excluded. In case of doubt or problem, contact your physician immediately.

Materials

- Foams: Polyethylene, OEKO-TEX STANDARD 100 certified
- Clamps and clips: Polyamide 66 - EPLAMID 66 IMP NC Q1B802, white colorant
- Box: Polypropylene, white colorant

Specifications

- Dimensions: Length 7.5 cm | Width 1.6 cm | Height 2.9 cm minimum and 3.9 cm maximum
- Weight: 11 grams

Kit Contents

- 2 Stop-uri Penile clamps
- 3 pairs of 4 clip sizes: 24 clips in total
- Storage box
- Instructions leaflet

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Fabricante : Le Taïman SAS
Produttore : Le Taïman SAS